

# Surface Disinfection

## Using CaviWipes: Wipe ▶ Discard ▶ Wipe

### Clean before disinfecting



1. Dispense CaviWipes towelette.



2. Clean surface with towelette to remove debris and bioburden.



3. Discard used towelette.

### Disinfect after cleaning



4. Dispense another CaviWipes towelette.



5. Disinfect pre-cleaned surface with towelette. Follow label instructions for appropriate contact times.



6. Discard used towelette.

### Use CaviWipes™ to clean and disinfect exterior surfaces of:

Hard, nonporous surfaces to prevent cross contamination, such as:

- Light fixtures
- Cabinets, chairs, tables and countertops
- Telephone and office machines
- Infant incubators

**Note: Follow standard precautions for personal protection as appropriate.**



CANADIAN PART NO.	U.S. PART NO.	PRODUCT DESCRIPTION	PACKAGING
11-1100	13-1100	CaviWipes – 160 wipes per can	12 canisters/case
11-1150	13-1150	CaviWipesXL – 66 wipes per can	12 canisters/case
11-1155	13-1155	CaviWipesXL – 50 wipes per box	6 boxes/case
13-1175	13-1175	CaviWipes canister wall bracket	12 brackets/case



**Metrex**  
Protecting People

800.841.1428 metrex.com